



**Locator with two
SMART™ switches**



**Locator with three
SMART™ switches**

West-Com's manual Locator Stations (Novus® and 2000-Series) are available in either two or three levels of staff location and can be used for a variety of functions. Caregivers who enter a room and press the locator Sealed Membrane Active Raised Tactile or SMART™ switch illuminate the dome light, indicating that staff is in the room. The Master Station displays the location, and the patient's call can be canceled by the appropriate staff level. When used with the West-Call® Novus® system, locators can be a signaling device for any purpose.

- ▶ Highly visible call LED to assure that the call has been placed; 500,000 hour life expectancy
- ▶ Easy-to-operate color-coded locator SMART™ switches
- ▶ Fire retardant and high impact molded thermoplastic
- ▶ Solid state circuitry
- ▶ Plug-in connectors for easy system connection/service
- ▶ Custom colors and text available

VERSIONS AND MODEL NUMBERS:

STATION TYPE	NOVUS®	NOVUS® VISUAL	HYBRID, ENDEAVOR & ODYSSEY
Locator with two SMART™ switches, Green and Orange (NV LOC/2 connects to NV-DOME or LDL2 only; LOC 2000/2 must connect to patient station)	NV-LOC/2		LOC-2000/2
Locator with three SMART™ switches, Green, Orange and Violet (NV LOC/3 connects to NV-DOME or LDL2 only)	NV-LOC/3		
Locator with two SMART™ switches, Green and Orange (NVV LOC connects to NV VDL/x)		NVV-LOC /2	
Locator with three SMART™ switches, Green, Orange and Violet (NVV LOC connects to NV VDL/x)		NVV-LOC/3	

- For use with the West-Call® Novus®, Odyssey and Endeavor systems
- Custom colors and text available at an additional cost

Operating Voltage	28 VDC
Power Consumption	1 Watt maximum
Size	4 1/2"H x 2 3/4"W x 1 1/4"D
Mounting	4" square backbox with 1 gang plaster ring, Raco 190 with 768 or equivalent. 2 1/8" minimum depth required. Normal mounting is <u>Vertical</u> .
Faceplate	Hospital grade ABS UL94V-0 molded thermoplastic with inlaid membrane
Faceplate Color	PMS 427; Light Grey Opaque
Membrane Color	.010 Lexan with hardcoat finish background color Grey (Pantone 4C). Copy color Warm Grey (Pantone 1C and 11C).
Termination	Plug-in connector allows rapid mass termination. Novus®: RJ-45; Odyssey, Hybrid and Endeavor: Panduit CE100F22-8
Cable Requirements	CAT 5e Cable from device to dome light when used with a Novus® NV LOC/2 or NV LOC/3; when using an LOC-2000/2 use 22/6 when connecting to a Novus® Hybrid Patient Station.
Certifications	UL-1069 for US and Canada, E139082

NOTE: This information should not be used for final design. Consult West-Com for total system requirements. Specifications subject to change without notice.



View West-Com's complete West-Call® line of healthcare communications solutions at www.westcall.com

West-Com Nurse Call Systems, Inc.
 2200 Cordelia Road • Fairfield, CA 94534 • 800-761-1180 • info@westcall.com

